



Metropolitan Diagnostic Imaging Group^{LLC}

IntegradWeb[®] PACS

Guide to Shortcuts

Action

Page/scroll the key image page	Use the control key while clicking the mouse or arrow keys
Pan while in Zoom mode	Use the middle mouse key roller
Zoom while in Pan mode	Use the middle mouse key roller
Open presets while in W/L mode	Use the middle mouse key roller
Get 1.00 zoom	Use the shift key
Refresh the study list	F5

Keyboard shortcut (system-supplied)

Action with user-configured keyboard shortcut

Jump to first image	home
Jump to last image	end
Jump to middle image	insert
Page fast	page up/page down
Patient Folder	F1
Triangulation	F2
Update the study	F11
Cine	C
Done	D or F12
Exit	X
Flip	F
Magnifying Glass	G
Mirror	9
Insert key image	I
Open key image preview	K
Edit key image template	ctrl+alt+P
Labels (annotations)	L
Measurements	M
Page/Scroll	P
Recommended Windowing	Q
Reset auto windowing	R
Rotate	0
Sharpen	S
Invert	V

Window/Level mode	W
Zoom mode	Z
Set zoom to 1	1
Set zoom to 1.5	5
Set zoom to 2	2
Set zoom to 2.5	ctrl+2
Set zoom to 3	3
Set zoom to 3.5	ctrl+3
Set zoom to 4	4
Undo the zoom	6
Next preset window	space bar
First page	home
Last page	end
Next page	up arrow
Previous page	down arrow
Page jump up	page up
Page jump down	page down
GUI editor	ctrl+alt+G
Layout editor	ctrl+alt+L
W/L editor	ctrl+alt+W
Display HTTP statistics	alt+H
Fast backward	shift+left or shift+down
Fast forward	shift+right or shift+up