



# Metropolitan Diagnostic Imaging Group<sup>LLC</sup>

## IntegradWeb<sup>®</sup> PACS

### Guide to Shortcuts

#### Action

Page/scroll the key image page .....	Use the control key while clicking the mouse or arrow keys
Pan while in Zoom mode .....	Use the middle mouse key roller
Zoom while in Pan mode .....	Use the middle mouse key roller
Open presets while in W/L mode .....	Use the middle mouse key roller
Get 1.00 zoom .....	Use the shift key
Refresh the study list .....	F5

#### Keyboard shortcut (system-supplied)

#### Action with user-configured keyboard shortcut

Jump to first image .....	home
Jump to last image .....	end
Jump to middle image .....	insert
Page fast .....	page up/page down
Patient Folder .....	F1
Triangulation .....	F2
Update the study .....	F11
Cine .....	C
Done .....	D or F12
Exit .....	X
Flip .....	F
Magnifying Glass .....	G
Mirror .....	9
Insert key image .....	I
Open key image preview .....	K
Edit key image template .....	ctrl+alt+P
Labels (annotations) .....	L
Measurements .....	M
Page/Scroll .....	P
Recommended Windowing .....	Q
Reset auto windowing .....	R
Rotate .....	0
Sharpen .....	S
Invert .....	V

Window/Level mode .....	W
Zoom mode .....	Z
Set zoom to 1 .....	1
Set zoom to 1.5 .....	5
Set zoom to 2 .....	2
Set zoom to 2.5 .....	ctrl+2
Set zoom to 3 .....	3
Set zoom to 3.5 .....	ctrl+3
Set zoom to 4 .....	4
Undo the zoom .....	6
Next preset window .....	space bar
First page .....	home
Last page .....	end
Next page .....	up arrow
Previous page .....	down arrow
Page jump up .....	page up
Page jump down .....	page down
GUI editor .....	ctrl+alt+G
Layout editor .....	ctrl+alt+L
W/L editor .....	ctrl+alt+W
Display HTTP statistics .....	alt+H
Fast backward .....	shift+left or shift+down
Fast forward .....	shift+right or shift+up